## Neosil® Silicone Scar Treatment Products

### Daily Rounds SCAR TREATMENT *Transparent Silicone Scar Sheeting*

- 1" Diameter — 32/box

### Daily SCAR TREATMENT *Transparent Silicone Scar Sheeting*

- 1.5" x 3" Clear Sheets — 8/box

### Daily SCAR TREATMENT *Fabric Silicone Scar Sheeting*

- 1.5" x 3" Flesh-tone Sheets — 8/box

### Daily SCAR TREATMENT FOR LARGE SCARS *Transparent Silicone Scar Sheeting*

- 1" x 5" Clear Sheets — 8/box

### Ultra SCAR TREATMENT *Transparent Silicone Scar Sheeting*

- 5" x 5" Clear Sheets — 3/box

### Daily SCAR SERUM *Liquid Silicone Lotion and Roll-On Applicator* — 0.4 fl. oz.

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## Silicone Scar Treatment Products

**Fades. Reduces. Hydrates.**

Ideal treatment for scars resulting from cuts, burns, surgeries and cosmetic procedures.

## Neosil® Scar Therapy Products Instruction for Use

Before use, wash and dry existing scar or newly healed wound area.

### Scar Sheets

1. Tear the paper packet and remove the scar sheet.
2. Bend the scar sheet to reveal the split in the clear protective liner.
3. Peel away the protective liner from the adhesive silicone gel.
4. Apply sheet with adhesive silicone gel side down, directly onto the scar.

For best results, the scar sheet should be worn each day for a minimum of 12 continuous hours.

### To Re-use the Scar Sheet

Clean the adhesive side of the scar sheet after use with mild soap and rinse well with clean water. Each sheet can be used for up to 7 days, then discard and replace with a new sheet.

### Scar Serum

1. Unscrew the cap from the roller applicator.
2. Apply the roller ball to the scar, gently massaging the scar and surrounding skin for 60-90 seconds.
3. Repeat this procedure 3-4 times daily.
4. Store at a temperature of 15°- 30°C (60°- 85°F).

### CAUTIONS

Do not use Neosil® products on open wounds, cracked or broken skin.

Seek immediate medical help if:

- Neosil® products are swallowed or ingested.
- Neosil® products come in contact with the eye or other mucous membranes.

Consult your doctor before using Neosil® products on children under six years of age. Keep away from children.

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